

# Technical data sheet

## Electro ejaculator



### **Product description**

The electro ejaculator consists of a rectal probe and a power control system which gives stepwise control over the applied voltage.

### **Application**

Electro ejaculation is applied for bulls with reduced libido or animals where semen collection with artificial vagina is not possible.

The probe is inserted into the rectum and placed above the prostate gland. By a series of short low-voltage pulses, the ejaculation is triggered.

## Properties

- Manual adjustable pulse/pause ratio.
  - Power output with 4 different types of curves enabling balanced setting according to the animals size and sensitivity.
  - Availability of different sizes of probes allows to choose the best fitting probe depending on the animals size and age.
  - The probes are optimally insulated, offering highest safety for the operator and the animal.
  - In case of emergency the power can immediately be switched off.
  - The electro ejaculator is supplied in a stable robust plastic carry case.
  - The device is equipped with a digital power control with microprocessor.
  - 3-digit digital display for the indication of the power output (display values from 0–20 with a resolution of 0.5 units).
  - 12 V extended power supply source.
- 
- For bulls two different probe sizes are available:
    - Ø: 2"/ 5.08 cm; length: 33 cm; 3 electrodes
    - Ø: 2.5"/6.35 cm; length: 40 cm; 3 electrodes
- 
- For rams one probe size is available:
    - Ø: 1"/ 2.54 cm; length: 16 cm; 3 electrodes
- 
- For small animals (dogs and cats) two different probe sizes are available:
    - Ø: 3.8"/ 0.95 cm; length: 6.35 cm; 3 electrodes
    - Ø: 1.2"/ 1.27 cm length: 6.35 cm; 2 electrodes

Electro ejaculator probes for bulls



Electro ejaculator probe for rams



Electro ejaculator probes for cats and dogs



## Order information

Electro ejaculator basic unit with case	<a href="#">REF. : 11900/0000</a>
Electro ejaculator probe, 2.5" for bulls	<a href="#">REF. : 11900/0025</a>
Electro ejaculator probe, 2" for bulls	<a href="#">REF. : 11900/0022</a>
Electro ejaculator probe, 1" for rams	<a href="#">REF. : 11900/0020</a>
Electro ejaculator probe, 3.8" for dogs	<a href="#">REF. : 11900/0509</a>
Electro ejaculator probe, 1.2" for dogs	<a href="#">REF. : 11900/0510</a>
Semen collection funnel and holder	<a href="#">REF. : 11900/0250</a>

Semen collection funnel and holder

